vista^{CTO4}

FOUR POST

UPGRADE KIT

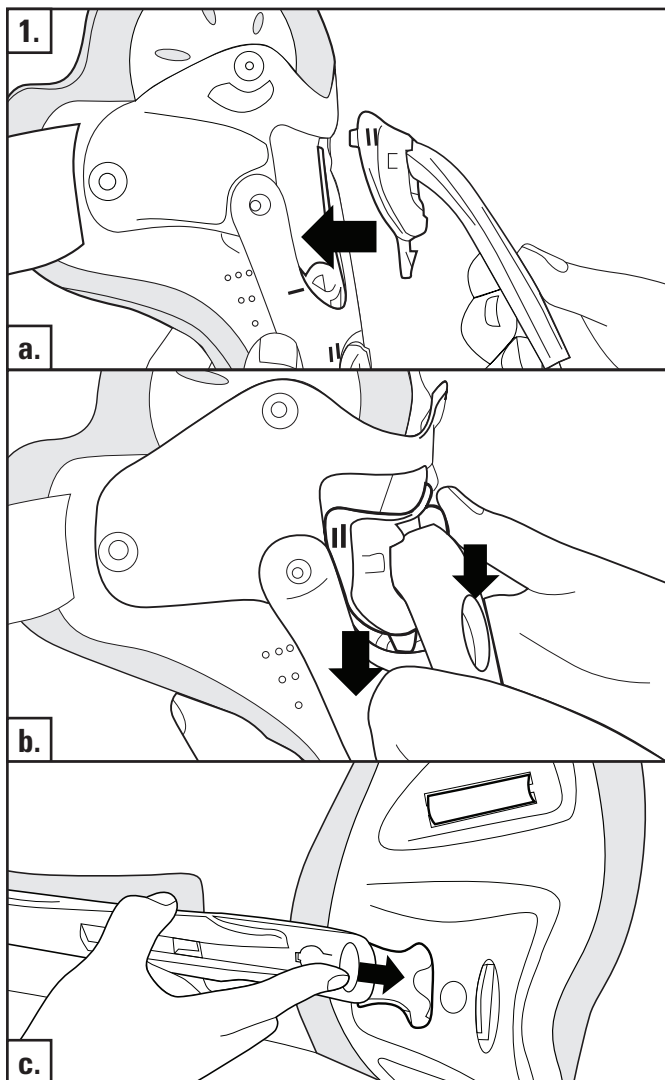
Aspen Medical Products

6481 Oak Canyon, Irvine, CA 92618-5202, USA

800-295-2776 • 949-681-0200

aspenmp.com

ENGLISH

STRUT APPLICATION

NOTE: If MultiPost collar has been applied, remove it and attach the posterior strut (see MultiPost instructions at www.aspenmp.com).

a. Align three connection posts with three openings in MultiPost back panel and push into position.

b. Push down causing strut support to lock into place. Insure that all three connectors are engaged.

c. Slide posterior strut into slot on lumbar belt posterior panel until it snaps into place.

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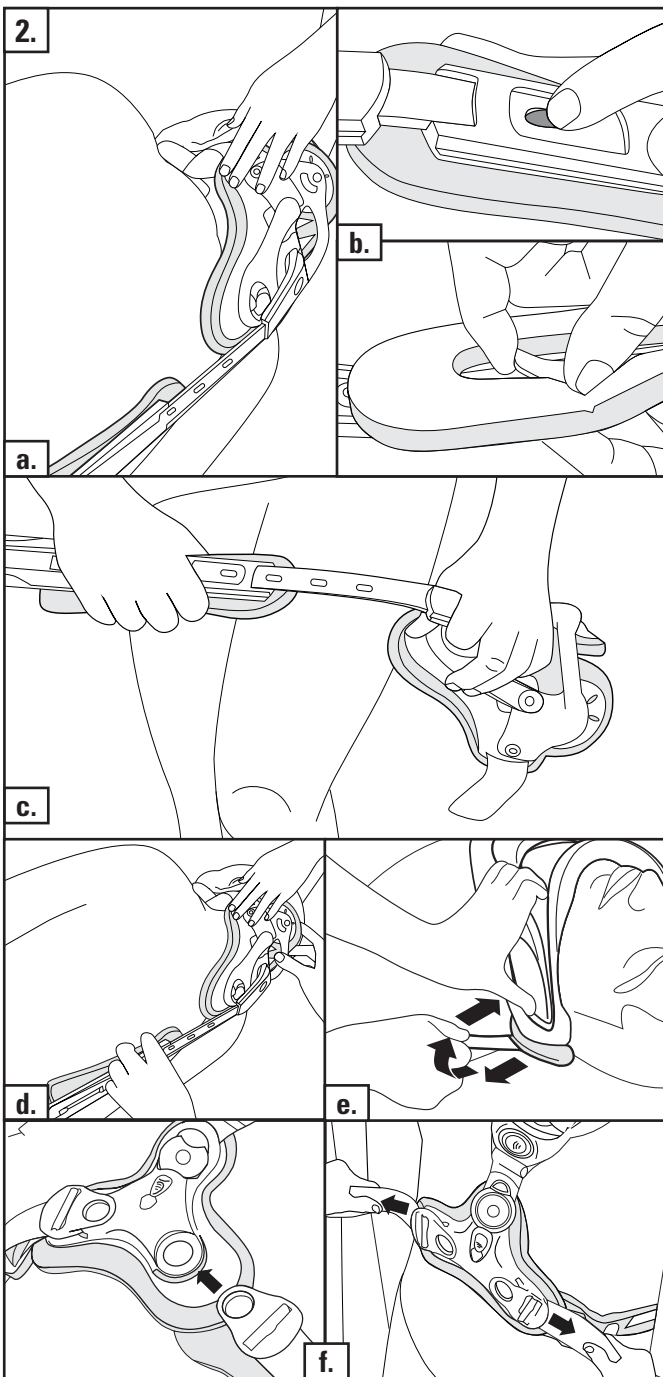
ENGLISH

CTO 4 POSTERIOR APPLICATION

NOTE: Maintain neutral alignment, and with proper support, log roll patient onto their side.

a. Position posterior assembly and collar back panel on patient.

b. To adjust the length push down on button. On backside, with other hand, pry back to fully release. Adjust strut up or down.

c. If necessary bend metal strut to account for kyphosis.

NOTE: Confirm adjustment buttons are secure following any adjustment.

d. Log roll patient back onto their back.

e. Tighten both collar straps as detailed in Vista MultiPost instructions.

f. Snap lumbar straps into vest and tighten as needed to achieve a snug, comfortable fit.

NOTE: For Vista CTO 4 application, see Vista CTO 4 instructions www.aspenmp.com.

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LIMITED WARRANTY

Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products shall have no obligation under this limited warranty in the event:

- (a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
- (b) The product is altered;
- (c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
- (d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.

This warranty gives you specific legal rights and you may have other rights which vary from state to state.

Warning: To ensure proper use, please review all material.

A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING