

PROACTIVE WEAR™

Suspension Sleeves



Shown in Natural

100 Series




Shown in Tan

200 Series



Shown in Golden Tan

300 Series

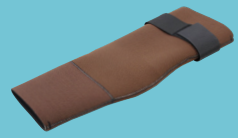

Shown in Dark Brown

400 Series

Colors

Natural

Tan

Golden Tan

Dark Brown

Black

ProActive® Wear Knee Sleeves for auxiliary suspension provide outstanding suction with no cracking, fraying, or running. The sleeves are contour-shaped to the knee for patient comfort and reduction of bunching in the popliteal area. Each sleeve is bonded with liquid neoprene and blind stitched, creating a smooth seam to assure a comfortable fit. Four styles and four natural skin tones to choose from.

Sizing Information

Size	0000	000	00	0	1	1.5	2	2.5	3	3.5	4	5
Circumference*	7-8.5	8-9.5	9-11	10.5-12	12.5-13	13.5-14	14.5-15	15.5-16	16.5-17	17.5-18	18.5-20	20.5-22

* Circumference measured in inches, 4" above knee center.

Donning Instructions

1. Fit knee sleeve over the prosthesis
2. Fold the top of the sleeve down
3. Position the residual limb in the socket
4. Grasp top edge of the sleeve and pull up gently
5. Position upper portion of sleeve on thigh for proper suspension and comfort

Care Instructions

Hand or machine wash gentle cycle for 5 to 10 minutes with warm water. Do not use bleach. Air dry.

Suspension Sleeve Materials **LATEX FREE**

100 Series

Exterior: Nylon

Interior: 3 mm Neoprene

200 Series

Exterior: Nylon

Interior: Nylon

300 Series

Exterior: Nylon


Interior:
Nylon Center;
3 mm Neoprene
Proximal and Distal

400 Series

Exterior:
Nylon with Strap


Interior:
Nylon Proximal and Center;
3 mm Neoprene Distal

Additional **PRO**TM Series Products

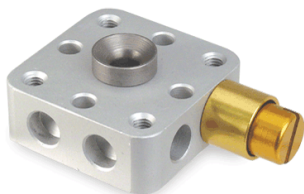

**Prosthetic
Locks**

IRS-XXX-XX

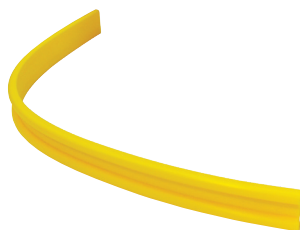


**Orthotic
Cutting Strip**

PL-CUTTINGSTRIP-XX



Adhesives

PF-CA2OZ-XXX

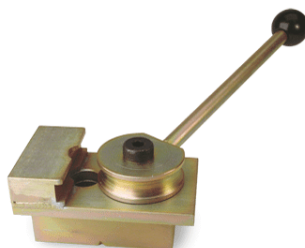

**Prosthetic
Vise**

PL-T04



**Bending
Iron**

PL-800



**Lamination
Plates**

1320-ADAPT, 1210-P

 **Cascade Orthopedic Supply, Inc.**

800-888-0865 | www.cascade-usa.com